



GRIP-LOK®

Suture-free fixation system for ARROW CVC/PICC catheters

AT A GLANCE

- safe
- easy application
- hypoallergenic
- breathable and latex-free to reduce the risk of allergic reactions and skin irritation

FIXATION MADE SAFE AND EASY

The Grip-Lok is one element of the global CVC / PICC concept developed for Teleflex for managing the risk of infection associated with catheterisation.

Grip-Lok is a suture-free fixation system which holds the catheter in place securely, whatever the access, and is suitable for use with any type of patient. The soft, pre-cut, hub-shaped component makes it easier to position the catheter and ensures perfect fixation and stabilisation.

- soft, flexible design for greater patient comfort
- improved catheter stabilisation during accidental vertical and horizontal infusion-line pulls
- easy to apply, inspect and adjust



Suture-free fixation Grip-Lok



The soft, pre-cut, hub-shaped component ensures perfect fixation and stabilisation



The soft, pre-cut, hub-shaped component also fits the catheter's additional fixation wings

**POSITIONING GRIP-LOK™, THE SUTURE-FREE SYSTEM
FOR ARROW CVC/PICC CATHETERS**



Prepare the skin in accordance with established practice.

Preparation or shaving may be required for better adhesion.

- ① Open the top flap and slide the Grip-Lok under the catheter hub. Position the hub on the area of adhesive designed for this purpose and press gently onto the adhesive.
- ② Lift and remove the backing film from one side while holding the Grip-Lok in place. Do the same on the other side of the backing film.
- ③ Lift and remove the backing film on the flap to expose the adhesive.
- ④ Re-close the flap on the catheter hub and press onto the adhesive. The self-gripping part of the flap can be repositioned, allowing the catheter to be inspected and adjusted.

After securing the Grip-Lok, finish dressing in accordance with best current practice. Change the transparent adhesive dressing using the stretch technique.

The information given on the application of Grip-Lok is no substitute for carefully reading the instructions for use.

GRIP-LOK®

REF.	DESCRIPTION	QTY
3303MCS-TA-1	suture-free fixation system for ARROW CVC/PICC Catheters	50

To view the Grip-Lok demo-video please visit the site www.zefon.com



Grip-Lok is a trademark of Zefon International.

DISTRIBUTED BY:

TELEFLEX MEDICAL HEADQUARTER EMEA, IRELAND · IDA Business Park · Athlone · Co Westmeath
Tel. +353 (0)9 06 46 08 00 · Fax +353 (0)14 37 07 73 · orders.intl@teflexmedical.com

UNITED KINGDOM Tel. +44 (0)14 94 53 27 61 · info.uk@teflexmedical.com

WWW.TELEFLEXMEDICAL.COM

94 10 93 - 00 00 01 · REV A · 02 11 02 · All data current at time of printing. Subject to technical changes without further notice.

teflex